HEALBOND

Description

1 components light-cure bonding system

Composition

Methacrylates and alcohol

Safety notes

Warnings

Healbond contains polymerizable methacrylates and alcohol which may irritate skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.

Avoid eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.

Avoid skin contact to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, immediately remove material with cotton and wash thoroughly with water and soap. In case of skin sensitization or rash, discontinue use and seek medical attention.

Avoid contact with oral soft tissues/mucosa to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water after the restoration is completed and expectorate/evacuate the water. If sensitization of mucosa persists, seek medical attention.

Precautions

Wear suitable protective eyewear, mask, clothing and gloves.

Protective eyewear is recommended for patients.

Contraindication

Healbond is contraindicated in patients with a history of allergic reaction to methacrylate resins or any other of the components.

Instructions for use

Isolation

Rubber dam is the recommended method if isolation.

Cavity preparation

Prepare the cavity with minimal tooth reduction. Margins should have a slight (0.5 to 1.0 mm) bevel placed in the enamel to increase the surface area for greater bond strength.

Pulp protection

Cavity floor very close to pulp should be covered with a thin layer of hard setting calciumhydroxide material or with an anti-microbial primer.

Instruction for use

Application of the etching gel

- Prepare tooth by cleansing the enamel surface to be etched with a slurry of pumice without oil or fluoride.
- Apply etching gel onto the enamel and dentine surfaces, beginning with the enamel bevels.
- Recommended time is 20 seconds.
- Rinse thoroughly with water. Do not contaminate with other fluids like saliva.
- Gently dry the surface with a water and oil-free airstream, but do not desiccate. A slightly wet dentine surface is important for a maximum adhesion of Healbond. The etched enamel bevel should have a chalky white appearance. If not, reapply the etching gel, following the previous points

Application of Healbond

- Apply immediately with agitation, an uniform coating of Healbond onto the prepared dentin and enamel surfaces for 30 seconds with a rubbing motion.
- Dry with oil free air for 10 seconds to evaporate the solvent base and to disperse the bonding homogeneously.
- Light-cure for 20 seconds and place the composite of your choice.

Storage and shelf-life

Store at a temperature not exceeding 23°C. When stored under such conditions, Healbond has a shelf-life of 24 months. Do not expose the product to sun-light or warm sources. Never store Healbond close to sparkles, open fire. Keep the products in a cool place but not in a fridge. Do not freeze.

Presentation & references

B-5 – 5ml bottle

Important notice

- Do not throw away the box and the instruction for use until the product is completely finished.
- High humidity could damage the packaging on which important instructions and legal information are printed.
- Storage at a too high temperature could cause to age prematurely and inhibit its polymerization.
- Storage at a too low temperature leads to the decomposition of the product which will render it completely unusable.

Disclaimer

These materials have been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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